
[EN] Tempal App Software User Manual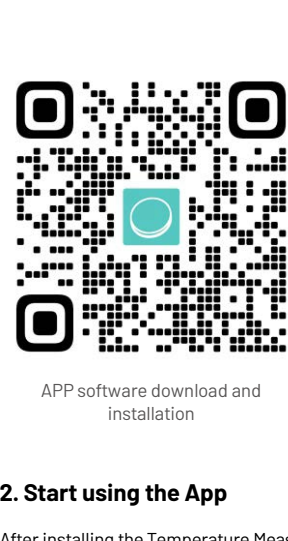

APP software download and installation

1. Tempal APP software download and installation

- Scan the QR code on the left -> Jump to the APP download page -> Click Download.
- Tempal APP supports devices:**
- Android devices: Android devices running Android 6.0 and above, supporting Bluetooth 4.0 and above versions.
 - Apple device: Apple devices running iOS14.0 and above, supporting Bluetooth version 4.0 and above.


2. Start using the App

After installing the Temperature Measurement App, click on the App icon on your phone's desktop to launch the App.

2.1 Search & Connectivity

2.1.1 Bluetooth permission: Enable the Bluetooth function of the cell phone to allow the Temperature Measurement App to access the Bluetooth permissions.

2.1.2 Device Connection: Remove the thermometer from the dormant compartment and bring it close to your phone. The APP will automatically search for nearby Bluetooth thermometers and connect to them, the temperature value will be displayed in the App after the "Connected Successfully" prompt appears.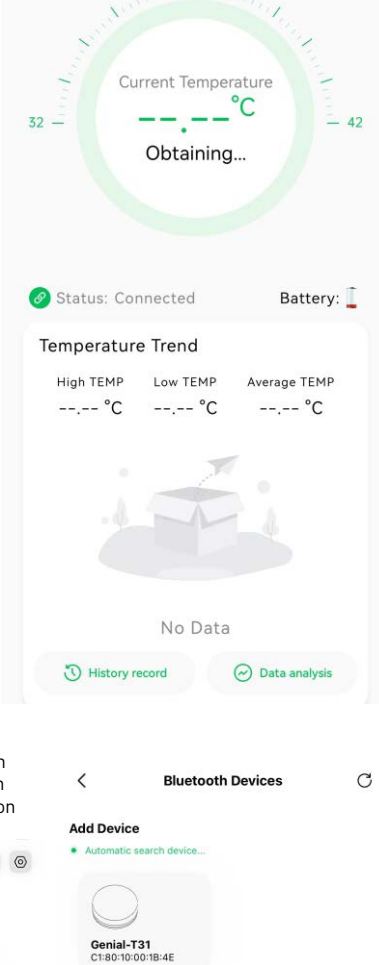

2.1.3 Device Search: If the app does not connect with the device automatically, or you want to connect with a new device. You can click the "Magnifying glass" icon button on the top right corner of the homepage to enter the device search page.

The top half of the search page has a list of searched devices, and the matched pair in the bottom half are devices that have been connected before. Clicking on the circular arrow button on the top right corner of the page indicates that you are searching for a new device. The device that has been connected will automatically disconnect the connected device and connect the new device as the default device, which will be the connected identity reset next time.


2.2 Temperature Alert Record

Click the second "Notification" icon button on the top right corner of the homepage to enter the temperature alert record page. Click the Select Date button on the upper right corner, select the date you want to check, and click OK to display the alert records of the selected date. The temperature, time and the temperature range of the warning are recorded here. As shown in the figure on the right.


2.3 System settings

Click the third "Gear" icon button on the top right corner of the homepage to bring up the setting page. You can switch the temperature unit between Celsius "C" and Fahrenheit "F" (default Celsius "C"), alert settings is on by default, if it is turned off, no alert will be given. you can set the alert temperature interval, when the body temperature is higher than the set maximum temperature and continues to exceed the set high temperature alert interval, the app will send out a notification and alert ringtone and vibration to alert the user. As shown in the figure on the right.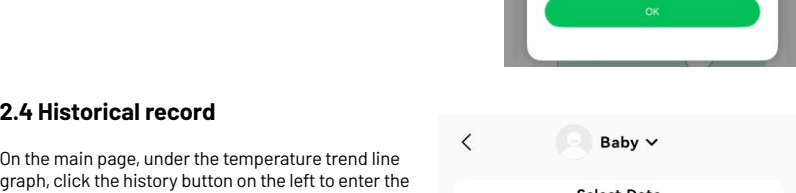


2.4 Historical record

On the main page, under the temperature trend line graph, click the history button on the left to enter the history page. At the top, click on the user's name or arrow to bring up the list of users, and click on switch to the user's history data. Select the date you want to check in the calendar, the temperature trend will be displayed in the form of a line graph. As shown in the figure on the right.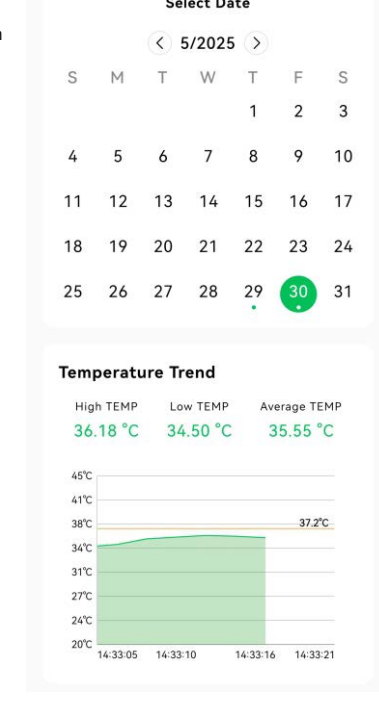


2.5 Data analysis

Under the Temperature Trend line graph on the main page, click the Data Analysis button on the right to enter the Data Analysis page. You can query the temperature data (including minimum temperature, maximum temperature and average temperature) for the current day, current week and current month. As shown in the figure on the right.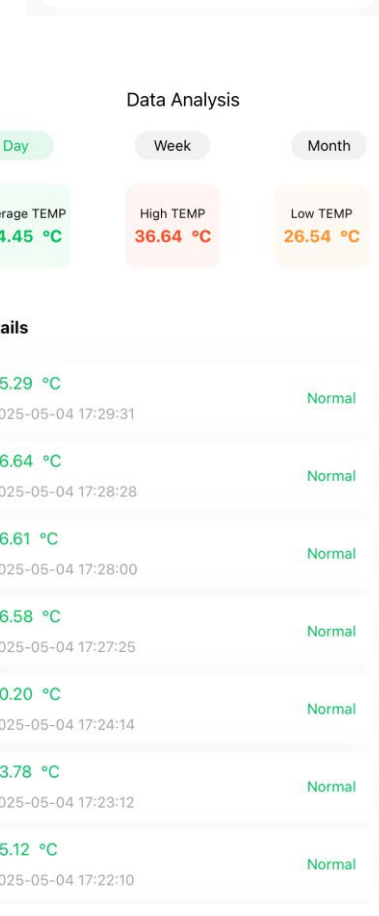

2.6 Warning sign


When the body temperature is higher than the set maximum temperature and continues to exceed the set warning interval, the app will automatically ring and vibrate to alert you. A warning bar will pop up on the homepage, if you need to remind again, click "Got it", a selection box will pop up, you can choose to be reminded after one hour, after three hours or not to be reminded in one hour, three hours or not to be reminded.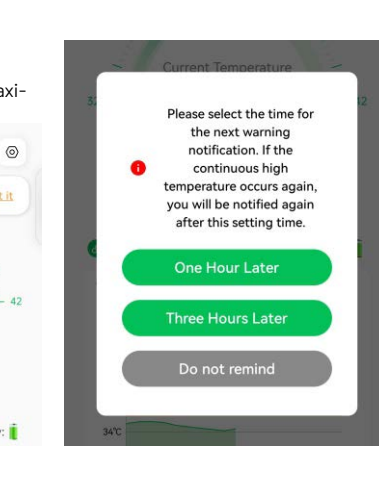

2.7 Out of Temperature Measuring Range APP Software Alerts


When the measurement temperature is lower than 20°C, the APP software will show the "Lo" prompt of out of measurement range. The interface is as follows.

When the measurement temperature is higher than 45°C, the APP software will show "Hi" to indicate that the measurement range is exceeded. The interface is as follows.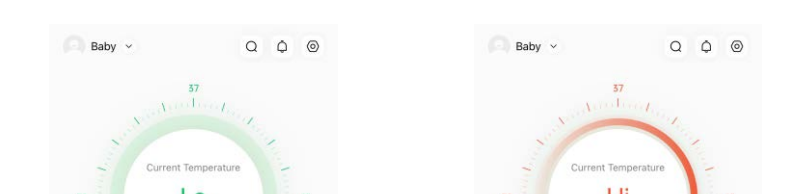

2.8 Add new member

Click the user name or the arrow next to the user name in the homepage, a user list will pop up. Click add new member, will bring up the user form, avatar can be selected according to personal preferences. Fill out the user's basic information can be submitted after. On the list of users, you can click on the switch, you can switch the measurement of the user as shown in the figure.

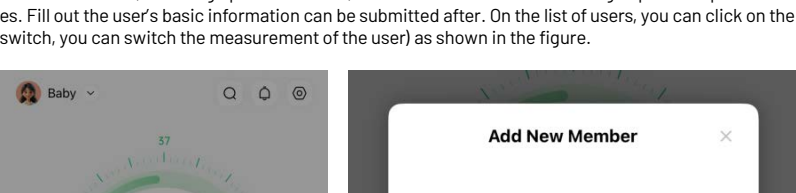3. Other notes

3.1 Other notes to measure temperature correctly prompts: Please follow the instructions for use of the body temperature sticker device to correctly apply the body under-arm area.

4. Common problems

4.1 Thermometers not found
If the device cannot be searched for, please check if the device has been taken out of the deep compartment and is emitting a blinking green light, if it is not emitting a blinking green light, the device may not have been connected to another cell phone. If you still can't find the device, please check if Bluetooth and location are turned on (Bluetooth connection requires location authorization). Whether the authorization is not allowed, please go to the phone settings, find the application settings, and find the device to give the relevant permissions.

4.2 Unable to connect device
Please check if the device is too far away from the cell phone. Please check if the body warmer still has power.

4.3 The temperature has exceeded the temperature point set for the high temperature alert and no warning bell is heard
Please check that the cell phone media sound is not turned down too low, or that another Bluetooth audio device is connected.

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